# Fleurcup

## Català

Full explicatiu

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# Fleurcup

## English User Guide

Thank you for purchasing Fleurcup, offering hygienic reusable protection for women during their menstrual cycle. Fleurcup is made in France.

Before using Fleurcup, we recommend that you read the present guide carefully.

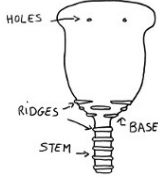
Keep the guide in a safe place for future reference.

If you have any additional questions, please visit our web site at [www.fleurcup.com](http://www.fleurcup.com) and/or contact us by e-mail: [serviceclient@fleurcup.com](mailto:serviceclient@fleurcup.com).

### Before using for the first time

Before using Fleurcup for the first time, **sterilize it**. To do this :

- Carefully wash your hands.
- Fill a pan with sufficient water for your Fleurcup to float on the surface.
- Bring the water to boil.
- Put your Fleurcup into the boiling water.
- Remove after 3 to 5 minutes.

### During your period

#### Inserting your Fleurcup

- Wash your hands and your Fleurcup carefully.
  - Pinch and fold your Fleurcup into a C shape.
  - Separate the labia, keep your Fleurcup folded and insert the cup into the vagina towards the direction of the small of your back, not towards your head, until the base of your Fleurcup is about one centimetre from the entry of the vagina.
  - Once your Fleurcup is inserted, it will unfold by itself.
  - To check that your Fleurcup is correctly positioned, hold the base of your Fleurcup (just above the stem) and rotate it. If it turns, it is correctly positioned. If not, re-position it.
- Other folding methods are presented on our web site [www.fleurcup.com](http://www.fleurcup.com) to enable you to find the method that suits you the best.

#### Removing your Fleurcup

Your Fleurcup should be emptied in relation to your flow and at least every 12 hours.

- Wash your hands carefully.
- Follow the stem to the base of your Fleurcup.
- Pinch the base of your Fleurcup or slide your index finger between the vaginal wall and the side of your Fleurcup in order to interrupt the suction effect.
- Remove your Fleurcup gently, taking care not to spill the contents.

#### Cleaning your Fleurcup

- Empty your Fleurcup.
  - Rinse your Fleurcup with cold drinking water.
  - Clean your Fleurcup, ideally with warm water and a perfume free, gentle soap so that this does not damage your Fleurcup, not forgetting the holes.
  - Rinse thoroughly.
- If you do not have access to drinking water, you can use toilet paper or a handkerchief to wipe your Fleurcup but not forgetting to wash it as soon as possible.
- You may also wish to carry a small bottle of water for rinsing your Fleurcup.
- You can now reinsert it.

### After the first few times

#### If your Fleurcup is not fully comfortable

- Your Fleurcup is positioned too high or too low. Try to use different folding techniques for insertion, these are explained with animations on our web site: [www.fleurcup.com](http://www.fleurcup.com).

- The stem is too long. In this case, remove your Fleurcup and cut a small length off the stem with a pair of scissors, taking care not to damage the

base of your Fleurcup (which would render it useless) until the stem is no longer a problem. Generally speaking, for optimal comfort, the stem should never go beyond the vagina.

#### If you observe leaks :

- Your Fleurcup is incorrectly positioned. If this is the case, after insertion, check that it has fully unfolded by rotating it or sliding your finger around the exterior of your Fleurcup.
- You have the Small Size Fleurcup but need the Large Size Fleurcup.

#### If you have difficulty in getting hold of your Fleurcup

- Don't panic, relax.
- Take up a squatting position or sitting on the toilet.
- Push as if for a bowel movement and your Fleurcup will descend naturally.
- You can then take hold of your Fleurcup and remove it.

### At the end of your period

- Clean your Fleurcup carefully.
  - Dry it.
  - Store it.
- Never store your Fleurcup in a plastic bag or an airtight box. Between cycles, remember to sterilise your Fleurcup.

### More about Fleurcup

- Fleurcup is made in France.
- Fleurcup is made 100% out of medical silicone.
- Fleurcup has a lifespan of several years if it is well maintained.
- The volume for the Small Size Fleurcup is approximately 20ml and the large size Fleurcup is approximately 29ml.
- Fleurcup can be used by virgins and by women with an IUD or a diaphragm.
- Fleurcup can be used at night and also during sports activities, even swimming.

### Important

- For any special uses of Fleurcup, please consult your gynaecologist before use.
- In the event of burning sensation, irritation, genital or vaginal inflammation or discomfort when going to the toilet or in the event of any other symptom, remove your Fleurcup immediately and consult your doctor.
- Fleurcup is not a means of contraception and does not offer protection against sexually transmissible diseases.
- It is not necessary to remove your Fleurcup to urinate or defecate.
- If you observe that your Fleurcup is damaged in any way, we recommend replacing it.
- The stem should not be used for pulling on to remove your Fleurcup. It is a guide to the base of your Fleurcup for removal.
- Remember to remove your Fleurcup prior to sexual penetration.
- Over time, your Fleurcup's colour may fade slightly, but this does not affect its performance.
- Keep out of the reach of children and animals.
- Your Fleurcup must not be washed in a dish washer or with corrosive products or with Aleppo soap.
- Fleurcup cannot be held responsible for prejudice arising due to incorrect care or use of your Fleurcup or for non respect of the user recommendations provided in the present guide and on its web site [www.fleurcup.com](http://www.fleurcup.com).

### Warning

Toxic Shock Syndrome (TSS) is a rare but occasionally fatal illness affecting men, women and children and is caused by a bacterial toxin. This illness has been associated with the prolonged use of tampons and their high levels of absorption.

It is characterised by symptoms which are similar to flu: a sudden, high fever, vomiting, diarrhoea, dizziness, skin rashes similar to sun burn, fainting... these symptoms can develop during the period or a few days afterwards. If you observe any of these symptoms while wearing the menstrual cup, remove it and consult a doctor immediately.

The use of the menstrual cup has never, to the present day, been associated with TSS.

If you have already suffered from TSS, do not use your Fleurcup.

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